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## English

**INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**SUITABLE FOR**

Due to the variety of products in the Thermo-med range, we have listed the most important indications; if you have purchased a Thermo-med product that hasn't been listed, please consult your dealer or medical centre. All of the orthoses serve as methods of protection for athletic activities, they are also indicated for Thermo compression, stabilisation, protection, oedema reduction, inflammation, etc.

**Depending on the product:**

**Ankle Supports:** Medio-Lateral stabilisation, immobilization, posture control.

**Knee Supports:** Rotular centring, prevention and pre-and post-operative treatments.

If the product is articulated: Medio-lateral stabilisation, flexo-extension control, immobilisation, hyperextension control.

**Corsets:** Lower back pain, rheumatic pain and for activities which require continuous exertion.

**Elbow supports:** Prevention of athletic injuries, epicondylitis.

**Wrist supports:** Stabilisation of radio-cubital inferior joint. If the product contains a palm splint: (a cook up splint). Immobilisation or limiting palmar flexion. Post-surgical treatments, posture control.

**With thumb splint:** Immobilisation of the metacarpal trapezoid, posture control in abduction and in maintaining the opposition of the thumb. Rhizarthrosis of the thumb, post-surgical treatments.

**FITTING INSTRUCTIONS**

**THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.**

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

**To fit observe the following aspects:**

**A-Ankle Supports: A / A1.**

How do you fit the plates?

**External malleolus plate:** Carries an identification on the front that should be placed according to whether the right foot or the left foot is being treated. It should be placed on the external part of the ankle support inserting it into its corresponding pocket.

**Internal malleolus plate:** Carries an identification on the front that should be placed according to whether the right foot or the left foot is being treated, place on the medial side of the ankle support, inserting it into its corresponding pocket.

**B-Knee Supports: (mold inner splints).**

1-Knee support with lateral springs.

2-Knee support with polycentric/ monocentric joint.

3-Knee support with flexo-extension.

The joints should remain parallel to each other, the mechanical shaft should exactly meet the anatomical shaft. In order to regulate the range of mobility of flexo-extension, before adapting regulate the degrees of flexion and extension desired, which will lift the lid of the joint. Blockage takes place at an extension of 0°, the flexo-extension can be regulated at intervals of 0-15-30-60-90 degrees, by inserting the lock in the different corresponding orifices represented on the mobility scale of the joint (close the lid). If a kneepad has been chosen that incorporates velcro pins, these must be the last to be tightened.

**C-Back supports.**

**D-Elbow Supports.**

**E-Wrist Supports:**

1-Palm splint wrist support.

2-Thumb wrist support.

**PRECAUTIONS**

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

**RECOMMENDATION- WARNINGS**

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community. The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed. For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

**MANUFACTURE - CHARACTERISTICS**

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

**RECOMMENDATIONS FOR PRESERVATION AND CLEANING**

Stick the velcros to each other, wash periodically by hand with a damp cloth and a neutral soap. To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or solvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product. Instructions for the extraction and fitting of lateral reinforcements inserted in knee supports – for ease in washing of products.

Pull at both ends of knee support until the hinge appears at the upper openings. Once product has been washed and completely dry, reinsert the reinforcements.

Precautions: In order to ensure the therapeutic effectiveness during normal use of knee supports, the reinforcements must never be handled or extracted from their correct position, except for washing or under specific orders from the prescribing doctor or orthopaedic technician.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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