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1H04Y

Dynamic carbon foot

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The dynamic carbon foot has been designed for patients with different degrees of activity, with unilateral or bilateral femoral amputation at different levels or tibial proximal, transtibial or distal amputation. It can also be used in prostheses after knee or hip disarticulation. The prosthesis is made for the right and left foot.

TECHNICAL CHARACTERISTICS

Code	1H04Y				
Foot dimensions	24	25	26	27	28
Foot length, mm	240	250	260	270	280
Articulated foot weight, g	490 ±20	500 ±20	540 ±20	570 ±20	680 ±20
Heel height, mm	18 mm				
Temperature range for use	From -40 °C to +40 °C				
Enables angular adjustment (of the whole prosthesis)	on the frontal and sagittal planes up to ±7.5°				
Warranty period	2 years				

DESCRIPTION


The dynamic foot is manufactured in six categories of rigidity, depending on the patient's weight. The category of rigidity of the prosthetic foot is determined by the rigidity of the support module that is fitted to the foot.

To ensure maximum comfort and depending on anthropometric characteristics, patient activity and the level of amputation, it is possible to use prosthetic feet in proximate categories of rigidity. In this case, the patient's weight must not exceed the maximum allowed for the foot.

Rigidity category	Patient weight, kg	Colour of the support module housing	Maximum patient weight, kg
2	50-65	Green	80
3	66-75	Red	100 *
4	76-85	Orange	100 *
5	80-100	Yellow	125
6	101-125	Blue	125

* For foot sizes 24 and 25 with rigidity category 3 and 4, the maximum patient weight is 80 kg.

NOTE: On surface B of pyramid 2, the rigidity category number is shown (see Fig. 1).

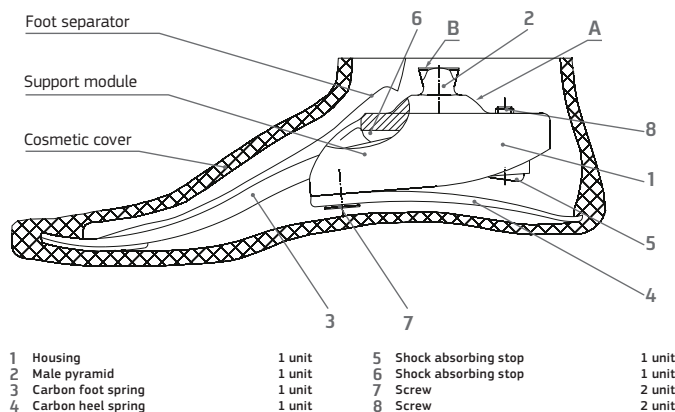DESCRIPTION OF THE STRUCTURE

The dynamic foot consists of a support module, cosmetic polyurethane cover and protective cover (P).

The foot's carbon fibre support module consists of aluminium housing 1 (Fig. 1), stainless steel pyramid 2, the foot's flexible carbon fibre elements 3 and heel 4, two polyurethane shock absorbing stops 5 and 6, and fixing screws 7 and 8.

The cosmetic cover is made from microcellular polyurethane reinforced with Kevlar fibre in the toe and heel areas.

The foot's protective cover is designed to eliminate possible annoying noises and to protect the inner part of the cosmetic cover from friction caused to it by the carbon module.

For attachment of the foot to other prosthetic modules, housing 1 features a spherical surface A bearing and pyramid 2. This connection enables angular adjustment on the frontal and sagittal planes in a range of not less than ±7.5°.

This dynamic foot is compatible with Orliman's modular lower-limb prosthetic systems.

MAINTENANCE

During use or cleaning, do not use talcum powder. Do not put the product into contact with corrosive or abrasive elements such as fresh or salt water, acids, dust, sand or other substances.

At least once a year, have the product checked by an orthopaedic technician for proper operation and wear. If you detect any unusual changes in the product's functioning or hear it making noises, stop using it immediately and notify an orthopaedic technician.

RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Elia - Apdo. de correos 49
C.P.: 46185 - La Pobla de Valbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

WARRANTY

2 años from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>
Signature and stamp of the orthopaedic establishment		For the warranty to be valid, it is necessary to complete these details.
<input type="text"/>		<input type="text"/>

LB-1-1H04Y
EDITION: 04/2015

ISO 13485

ISO 9001