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English

INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATION

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

SUITABLE FOR

Post-injury immobilisation, functional recovery, instability, prevention of sports injuries, fasciitis plantar.

FITTING INSTRUCTIONS

THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

To fit observe the following aspects:

To fit the VALFEET (XR OR AIR) place it into your footwear and release the 4 velcro straps. Once the VALFEET has been fitted to the footwear, separate the two valves so that you can comfortably put your foot into the VALFEET. Then push the foot back and adjust the closure of the footwear. We recommend you use footwear with closing or adjustment systems (laces, velcro straps). Do not use moccasins.

To ensure suitable and comfortable fitting, first attach the back straps and then the front ones. To remove your foot from the VALFEET release the straps and separate the valves.

Both Valfeet can be used with or without the insole of your footwear according to your personal preference. If having fitted Valfeet you do not obtain the desired level of comfort, remove the insole of your footwear.

Both Valfeet models have a set of screws in the packaging that allows flexo-extension movement blocking, if necessary. To attach the screws, remove the internal pads, drill under the joint axes between the valves and the sole and insert the screws. Replace the pad over the velcro.

Valfeet AIR:

The fitting procedure is the same as that for Valfeet XR, but there is an additional inflation system. Inflation system: it contains an air chamber that allows compression to be increased and the tibiotarsal articulation to be stabilised.

The orthosis includes an inflation and deflation pump. To inflate, place the end of the inflation pump into the valve of the air chamber at the top and inflate as required. To deflate use the other end of the pump and deflate as required.

PRECAUTIONS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

RECOMMENDATION-WARNINGS

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

MANUFACTURE-CHARACTERISTICS

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Hand wash periodically in warm water with neutral soap. To dry the orthosis, use a dry towel to absorb the water as much as possible and leave it to dry at room temperature. Do not hang out and do not expose to direct heat sources like heaters, radiators, direct exposure to the sun etc. Do not use alcohol, creams or liquid solvents during use or cleaning.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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