3H02C - 3H02T

› Tubo con adaptador hembra

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3H02C - 3H02T

› Tube with female adapter

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The tube with female adapter is used in tibial and femoral prostheses and after hip disarticulation. It compensates for the shortening of the amputated part of the leg or thigh, ensuring adjustable connection with other prosthetic modules.

TECHNICAL CHARACTERISTICS

Code	3H02T/200K	3H02T/400K	3H02C/200K	3H02C/400K
Item length, mm	215	415	216	416
Item weight, g	200	400	200	400
Patient weight, kg, no higher than	150		125	
Adapter 1 material	Titanium		Stainless steel	
Tube 2 material	Aluminium			
Outer diameter of the tube	30 mm			
Temperature range for use	From -40 to 40°C			
Angular adjustment of the whole prosthesis on the frontal and sagittal planes	From -7.5 to 7.5°			
Warranty period	2 years			


DESCRIPTION OF THE STRUCTURE

The tube with adapter consists of pyramid adapter (1) and tube (2) (Fig. 1). The connection of adapter 1 and tube 2 is carried out by tightening clamp B with screw 5.

Adapter 1 features a spherical surface A female bearing and is supplied with two pairs of screw 3 and 4 of 12 mm and 16 mm in length for connection to the modules of the prosthesis. On the prosthesis, this connection enables angular adjustment on the frontal and sagittal planes in a range of 7.5°.

Tube 2 is made from aluminium and has the shape of a hollow cylinder with outer diameter of 30 mm. It is designed to be fitted to modules that have similar elements to a fixing clamp with an inner diameter of 30 mm. On the prosthesis, this connection enables horizontal rotations to be carried out at the desired angle.

The tube with adapter is compatible with Orliman's modular lower-limb prosthetic systems.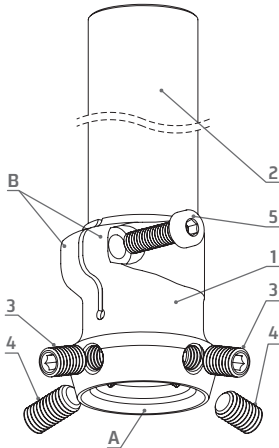


1 Female pyramid adapter	1 unit	4 M8 screw, L=16 mm	2 units
2 Tube	1 unit	5 Screw	1 unit
3 M8 screw, L=12 mm	2 units		

FITTING THE ITEM TO THE PROSTHESIS

The tube with adapter is sufficiently long to enable it to be trimmed to the required length according to the assembly needs. In this case, deformation of the tube should be avoided; the cutting surface should be smooth and form a right angle to the tube's longitudinal axis, and the outer edge of the tube must not have burrs.

When connecting the support module to other modules of the prosthesis, screws 3 and 4 should be fixed according to Figure 2 as follows: the 12 mm long screws should be placed where the space between the ring housing and the pyramid is smaller, and the 16 mm long screws, from the opposite side. It is necessary to ensure that the screws protrude from the outer surface of the tube adapter.

During final assembly of the prosthesis, it is recommended to apply Loctite 243 (or 241) threadlocker to the threads of screws 3 and 4 and tighten to 15 Nm.

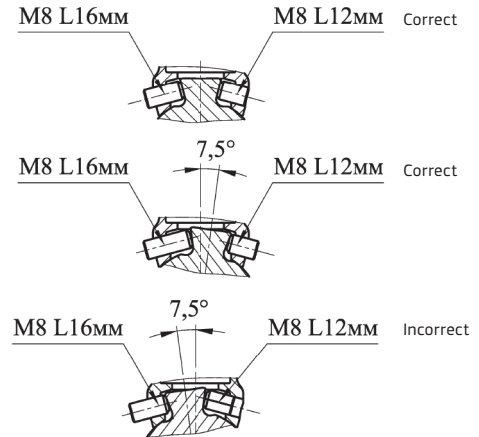RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49
C.P.: 46185 - La Pobla de Valibona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

WARRANTY

2 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-3H02

EDITION: 04/2015

ISO 13485

ISO 9001