4H04A - 4H04C - 4H04T

Adaptador piramidal doble hembra

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4H04A - 4H04C - 4H04T

› Double female pyramid adapter

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The double female pyramid adapter is used in lower-limb prostheses and serves to connect and adjust the modules, compensating for the shortening of the amputated part of the leg or thigh.


TECHNICAL CHARACTERISTICS

Code	4H 04T/35	4H 04C/35	4H 04A/35	4H 04T/45	4H 04C/45	4H 04A/45	4H 04T/60	4H 04C/60	4H 04A/60	4H 04T/75	4H 04C/75	4H 04A/75	4H 04T/90	4H 04C/90	4H 04A/90
Length, L, mm	35			45			60			75			90		
Housing material	Titanium	Stainless steel	Aluminium	Titanium	Stainless steel	Aluminium	Titanium	Stainless steel	Aluminium	Titanium	Stainless steel	Aluminium	Titanium	Stainless steel	Aluminium
Item weight, g, no higher than	90	135	85	97	146	103	107	164	111	117	181	122	127	197	135
Patient weight, kg, no higher than	150	125	100	150	125	100	150	125	100	150	125	100	150	125	100
Temperature range for use	From -40 °C to 40 °C														
Angular adjustment of the whole prosthesis on the frontal and sagittal planes	From -7.5 to +7.5°														
Warranty period	3 years														


DESCRIPTION OF THE STRUCTURE

The pyramid adapter (Fig. 1) consists of housing 1 with two spherical support surfaces A, two pairs of screws 2 and 3 of 12 mm and 16 mm in length and the support surfaces. This connection enables angular adjustment with other modules of the prosthesis on the frontal and sagittal planes in a range of -7.5 to +7.5°.

The pyramid adapter is compatible with Orliman's modular lower-limb prosthetic systems.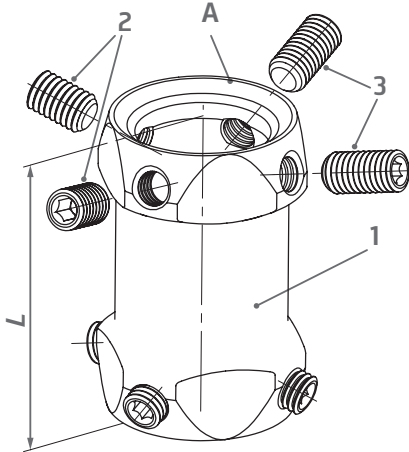

- 1 Housing 1 unit 3 M8 screw, L=16 mm 4 units
2 M8 screw, L=12 mm 4 units

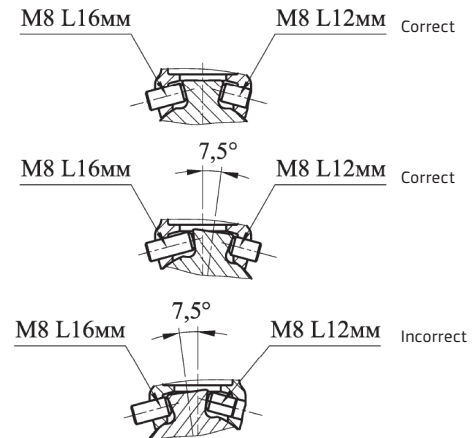During final assembly of the prosthesis, it is recommended to apply Loctite 243 (or 241) threadlocker to the threads of screws 2 and 3 and tighten to 15 Nm.

RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaua - Apdo. de correos 49
C.P.: 46185 - La Pobla de Vallbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com - Exports email: export@orliman.com - www.orliman.com

WARRANTY

3 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-4H04

EDITION: 04/2015

ISO 13485

ISO 9001