4H06A

Adaptador piramidal macho, tubo Ø30

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4H06T

› Male pyramid adapter, Ø30 tube

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The pyramid adapter is used in lower-limb prostheses and serves to connect and adjust the modules, compensating for the shortening of the amputated part of the leg or thigh.

TECHNICAL CHARACTERISTICS


Code	4H06T/50K	4H06T/77K
Length, L	50	77.4
Item weight	67	104
Patient weight, kg, no higher than	150	
Material	Titanium	
Outer diameter of the tube, mm	30	
Temperature range for use	From -40 to +40°C.	
Angular adjustment of the whole prosthesis on the frontal and sagittal planes	From -7.5 to +7.5°	
Warranty period	2 years	

DESCRIPTION OF THE STRUCTURE

The pyramid adapter (Fig. 1) features, on one side, spherical surface bearing B and pyramid A, and on the other side, cylindrical tube C with 30 mm diameter. The connection between the outer spherical surface and the pyramid enables angular adjustment of the whole prosthesis on the frontal and sagittal planes in a range of -7.5 to +7.5°.

The tube is designed to be fitted to the modules that have similar elements to a fixing clamp with an inner diameter of 30 mm. This connection enables horizontal rotations to be carried out at the desired angle.

The pyramid adapter is compatible with Orliman's modular lower-limb prosthetic systems.

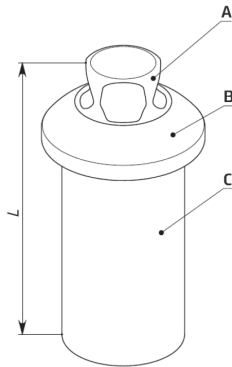FITTING THE ITEM TO THE PROSTHESIS

ATTENTION: to avoid annoying friction noises, there should be no contact between the adapter pyramid and the housing of the corresponding part.

The fitting of the adapter with the pyramid to the corresponding units with clamp and inner cylindrical surface should be carried out until the stop in the corresponding unit.

RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49
C.P.: 46185 - La Pobla de Vallbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com - Exports email: export@orliman.com - www.orliman.com

WARRANTY

2 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

<input type="text"/>

LB-1-4H06T

EDITION: 04/2015

ISO 13485

ISO 9001