4H09A

Adaptador piramidal hembra roscado

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4H09A

› Threaded female pyramid adapter

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION


The pyramid adapter is used in lower-limb prostheses and serves to interconnect and adjust the modules, compensating for the shortening of the amputated part of the leg or thigh.

TECHNICAL CHARACTERISTICS

| | |
|--|---|
| Code | 4H09A |
| Item length | 100 mm |
| Item weight | 180 g |
| For patients weighing: | up to 100 kg |
| Material | Aluminium |
| Temperature range for use | From -40 °C to +40 °C |
| Enables angular adjustment (of the whole prosthesis) | on the frontal and sagittal planes at an angle of not less than $\pm 7.5^\circ$ |
| Warranty period | 3 years |

DESCRIPTION OF THE STRUCTURE


Housing 1 of the threaded pyramid adapter (Fig. 1) features female spherical surface A and is supplied with two pairs of screw 2 of 12 mm and two pairs of screw 3 of 16 mm for connection to the modules of the prosthesis. On the prosthesis, this connection enables angular adjustment on the frontal and sagittal planes in a range of not less than $\pm 7.5^\circ$. The pyramid adapter is compatible with Orliman's modular lower-limb prosthetic systems.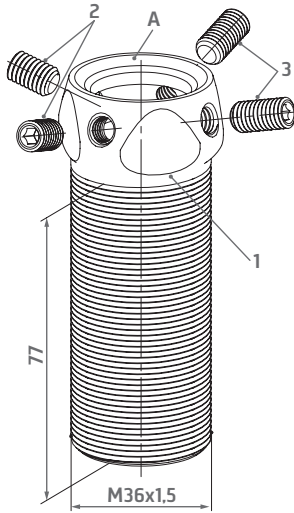

- 1 Housing 1 unit
2 M8 screw, L=12 mm 2 units
3 M8 screw, L=16 mm 2 units

FITTING THE ITEM TO THE PROSTHESIS

When connecting the pyramid adapter to other modules of the prosthesis, the short screws should be placed where the space between the ring and the pyramid is smaller, and the longer ones, where the space is greater (see Fig. 2). During final assembly, it is recommended to apply Loctite 241 threadlocker to screws 2 and 3 and tighten them with a torque of 15 Nm.

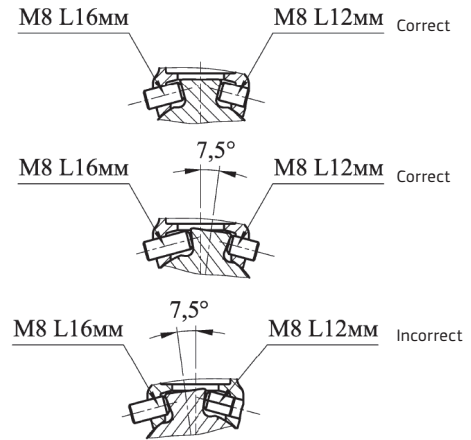RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician. When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community. In knee joints, there is risk of entrapment, use caution. As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses. ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Elia - Apdo. de correos 49
C.P.: 46185 - La Pobla de Valbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com - Exports email: export@orliman.com - www.orliman.com

WARRANTY

3 años from purchase

| | | |
|--|----------------------|--|
| Reference | Batch no. | Date of purchase |
| <input type="text"/> | <input type="text"/> | <input type="text"/> |
| Signature and stamp of the orthopaedic establishment | | For the warranty to be valid, it is necessary to complete these details. |
| <input type="text"/> | | <input type="text"/> |

LB-1-4H09A

EDITION: 04/2015

ISO 13485

ISO 9001