# 4H12AC - 4H12C - 4H12T

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ISO 9001

# 4H12AC - 4H12C - 4H12T

## › Threaded male pyramid insert for prong

### INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,  
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

### APPLICATION

The pyramid adapter is used with anchorings or rotatable lamination flanges in leg or thigh prostheses. The pyramid adapter is fitted to components that have an M36x1.5-6H internal thread and a clamp to fix ring 1, for example, 7H13CK or 7H13TK, and serves as the adjustable connection of the indicated components with other modules of the prosthesis.

### TECHNICAL CHARACTERISTICS


Code	4H12AC	4H12C	4H12T
Housing material	Aluminium / Stainless steel	Stainless steel	Titanium
Item weight, g	62	84	48
Patient weight, kg, no higher than	100	125	150
Temperature range for use	From -40 to +40°C		
Angular adjustment of the whole prosthesis on the frontal and sagittal planes	From -7.5 to +7.5°		
Minimum adjustment range of the whole prosthesis on the horizontal plane with respect to the sagittal plane	From -15° to +15°		
Warranty period	2 years		

### DESCRIPTION OF THE STRUCTURE

The pyramid adapter (see Fig. 1) features spherical surface bearing A and pyramid B for connection with other modules of the prosthesis. This connection enables angular adjustment on the frontal and sagittal planes in a range of -7.5° to +7.5°.

The back of ring 1 has a 9 mm high external thread D. Adjustment of the pyramid adapter on the horizontal plane is carried out by unscrewing it from the stop.

The pyramid adapter is compatible with Orliman's modular lower-limb prosthetic systems.

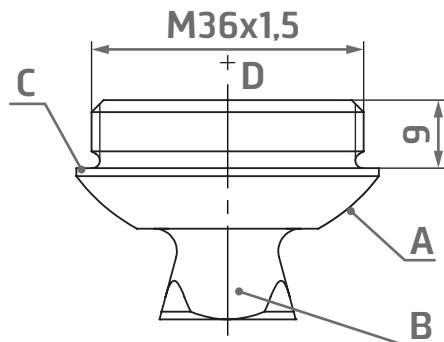### FITTING THE ITEM TO THE PROSTHESIS

The connection of the pyramid adapter to the component is carried out by screwing the support onto the M36x1.5 thread and fixing it with the clamp. Adjustment on the horizontal plane is carried out by turning ring 1 on the thread.

ATTENTION: The allowable space between the support and edge C of the ring with pyramid should not exceed 0.5 mm.

RECOMMENDATION: Highly active patients are recommended to use models 4H12T or 4H12C.

### RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed.

Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

### WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.  
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49  
C.P.: 46185 - La Pobla de Valldona Valencia - España (Spain)  
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33  
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

### WARRANTY

2 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-4H12

EDITION: 04/2015

ISO 13485

ISO 9001