# 4H14C - 4H14T

## Adaptador piramidal excéntrico macho-hembra

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# 4H14C - 4H14T

## Male-female double offset adapter

### INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,  
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

### APPLICATION

The double offset adapter is used in lower-limb prostheses and serves to connect modules that have a spherical surface bearing and a pyramid, and modules with spherical inner support surface and M8 fixing screws. In addition, on the adapter, the support surfaces are displaced by 11 mm.

### TECHNICAL CHARACTERISTICS


Code	4H14C	4H14T
Ring material	Stainless steel	Titanium
Item weight, g	186	114
Patient weight, kg, no higher than	125	150
Temperature range for use	From -40 to +40°C	
Angular adjustment of each support surface of the whole of the prosthesis on the frontal and sagittal planes	From -7.5 to +7.5°	
Warranty period	2 years	

### DESCRIPTION OF THE STRUCTURE

For connection to the modules of the prosthesis, pyramid adapter 1 (see Fig. 1) features two support surfaces: on one side, female inner spherical surface D with two pairs of screws 2 and 3 of 12 mm and 16 mm and, on the opposite side, outer spherical surface A with pyramid B. The displacement of the centre of spherical surface A with respect to the centre of spherical surface D is equal to 11 mm and the angle of the axis of pyramid B is 10°.

On the prosthesis, each of these connections can carry out angular adjustment on the frontal and sagittal planes in a range from -7.5 to +7.5°.


The double offset adapter is compatible with Orliman's modular lower-limb prosthetic systems.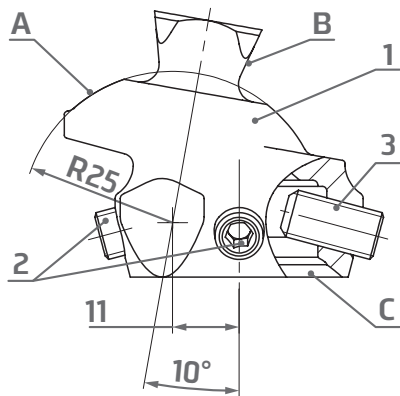

- |   |                             |         |
|---|-----------------------------|---------|
| 1 | Ring with eccentric pyramid | 1 unit  |
| 2 | M8 screw, L=12 mm           | 2 units |
| 3 | M8 screw, L=16 mm           | 2 units |

### FITTING THE ITEM TO THE PROSTHESIS

During connection of the adapter to other modules through spherical surface D, the screws should be placed according to Figure 2 as follows: the 12 mm long screws should be placed where the space between the adapter ring and the pyramid is minimal, and the 16 mm long screws, from the opposite side. In addition, there should be no direct contact between the pyramid and the housing of the adapter.

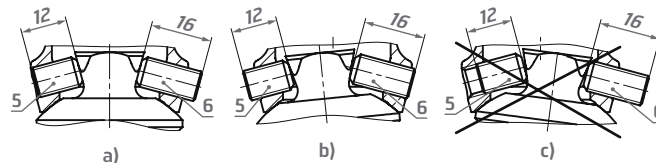- a), b) - correct connection of the module  
c) - incorrect connection of the module

During final assembly of the prosthesis, it is recommended to apply Loctite 243 (or 241) threadlocker to the threads of screws 2 and 3 and tighten them with a torque of 15 Nm.

### RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

### WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.  
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49  
C.P.: 46185 - La Pobla de Vallbona Valencia - España (Spain)  
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33  
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

### WARRANTY

2 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>
Signature and stamp of the orthopaedic establishment		For the warranty to be valid, it is necessary to complete these details.
<input type="text"/>		<input type="text"/>

LB-I-4H14

EDITION: 04/2015

ISO 13485

ISO 9001