7H01AK - 7H01CK - 7H01TK

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7H01AK - 7H01CK - 7H01TK

4-hole rotatable female pyramid

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The 4-hole rotatable female pyramid is used in lower-limb prostheses and serves to connect modules and sockets with flat supporting surfaces and four M6 threaded holes, and modules with outer spherical surface bearings and pyramids. The 4-hole rotatable male pyramid enables angular adjustment on three planes.

TECHNICAL CHARACTERISTICS

Code	7H01AK	7H01CK	7H01TK
Distance between centres A, mm	36	36	36
Item weight, g	101	186	125
Material of base 1 of bushing 4	Aluminium	Stainless steel	Titanium
Patient weight, (kg) no higher than	100	125	150
Temperature range for use	From -40 to +40°C		
Angular adjustment of the whole prosthesis on the frontal and sagittal planes	From -7.5 to +7.5°		
Minimum adjustment range of the whole prosthesis on the horizontal plane with respect to the sagittal plane	From -15° to +15°		
Warranty period	2 years		


DESCRIPTION OF THE STRUCTURE

The 4-hole rotatable female pyramid consists of base 1 (see Fig. 1) which features an M36x1.5 thread and four holes for fastening to the rigid socket, as well as ring 4 screwed to base 1 by means of the M36x1.5 thread.

Bushing 4 features female spherical surface bearing B and is supplied with two pairs of screws 5 and 6 of 12 mm and 16 mm in length to connect the socket to other modules of the prosthesis. This connection enables angular adjustment on the frontal and sagittal planes in a range of -7.5° to +7.5°.

Adjustment of bushing 4 on the horizontal plane is carried out by unscrewing it along the thread from the stop. Fixing bushing 4 on thread M36x1.5 is carried out by tightening slot B with screw 3 through bolt 2.

The 4-hole male pyramid is compatible with Orliman's modular lower-limb prosthetic systems.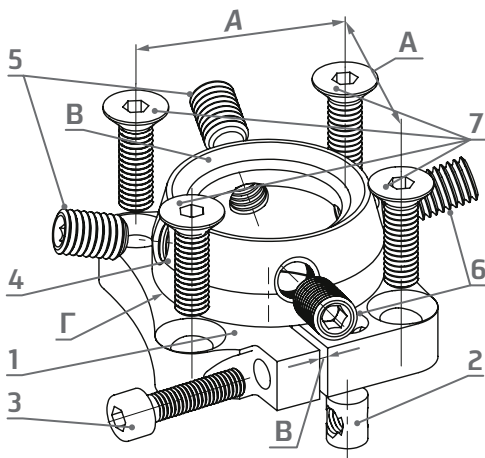

1 Base	1 unit	5 M8 screw, L=12 mm	2 units
2 Bolt	1 unit	6 M8 screw, L=16 mm	2 units
3 M5 screw	1 unit	7 M6 screw	4 units
4 Bushing	1 unit		


FITTING THE ITEM TO THE PROSTHESIS

To connect the 4-hole rotatable male pyramid to the socket support, screws 7 are used as shown in Figure 2 - the same ones that protrude 9 to 11 mm from the fixing surface.

The connection of base 1 of bushing 4 is carried out by screwing base 1 onto the M36x1.5 thread and fixing it with screw 3 of the retaining clamp. Adjustment on the horizontal plane is carried out by turning bushing 4 on the thread.

ATTENTION: The allowable space between the base and edge D of bushing 4 should not exceed 0.5 mm.

ATTENTION: During final assembly of the prosthesis, it is recommended to apply Loctite 241 threadlocker to the threads of screws 7 and respect the following screw tightening sequence: 3 and 7. Firstly, tighten the screws 7 that are furthest away from slot B with a torque 14 Nm, then screw 3 to 8 Nm and finally tighten those that are closest to slot B, as shown in Fig. 1. During assembly, through the spherical surface, the screws should be placed according to Figure 3 as follows: the 12 mm-long screws should be placed where the space between the adapter-bushing and the pyramid is smallest, and the 16 mm-long screws, from the opposite side. In addition, there should be no direct contact between the pyramid and the bushing housing.

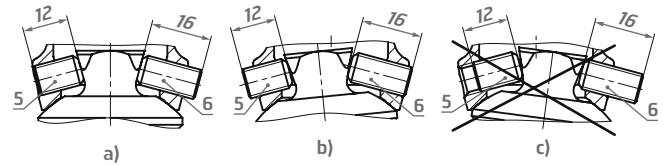a), b) - correct connection of the module

c) - incorrect connection of the module

During final assembly of the prosthesis, it is recommended to apply Loctite 243 (or 241) threadlocker to the threads of screws 5 and 6 and tighten them to 15 Nm.

RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed.

Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49
C.P.: 46185 - La Pobla de Valbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

WARRANTY

2 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-7H01

EDITION: 05/2015

ISO 13485

ISO 9001