# 7H13AK - 7H13CK - 7H13TK

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# 7H13AK - 7H13CK - 7H13TK

## 4-hole adapter

### INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,  
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

### APPLICATION

The 4-hole adapter is used in lower-limb prostheses and serves to fasten modules and sockets with flat supporting surfaces and four M6 threaded holes and M36x1.5-6g threaded cores.


### TECHNICAL CHARACTERISTICS

Code	7H13AK	7H13CK	7H13TK
Size □A, mm	36	36	36
Item weight, g	54	90	63
Base material	Aluminium	Stainless steel	Titanium
For patients weighing, kg	up to 100	less than 125	less than 150
Temperature range for use	From -40 to +40°C.		
Warranty period	2 years		

### DESCRIPTION OF THE STRUCTURE

The 4-hole adapter consists of base 1 (see Fig. 1) which features an M36x1.5 thread and four holes for fastening to the socket support. Height H of base 1 is 10 mm. To fasten the modules by means of the thread to base 1, it features through slot B, which is tightened with screw 3 through bolt 2.

The 4-hole adapter is compatible with Orliman's modular lower-limb prosthetic systems.

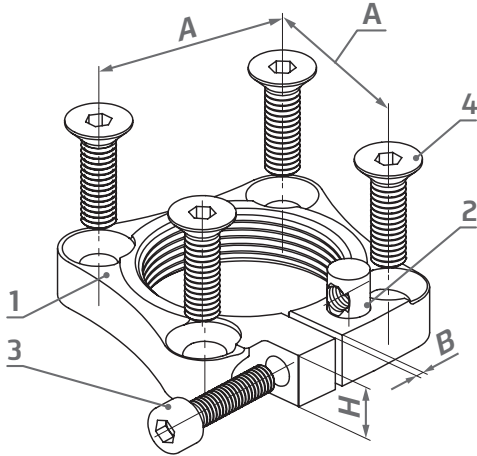- |        |        |            |         |
|--------|--------|------------|---------|
| 1 Base | 1 unit | 3 M5 screw | 1 unit  |
| 2 Pin  | 1 unit | 4 M6 screw | 4 units |

### FITTING THE ITEM TO THE PROSTHESIS

When adjusting the cores to the M36x1.5 thread of base 1, it is necessary to ensure that the length of the screwing of the pair is not less than 9 mm, in addition, to carry out the adjustment, it is possible to reduce the length of screwing to 8.5 mm. The fastening of the core to thread B is carried out with the retaining clamp of base 1 with screw 3.

The 4-hole adapter is fastened to the socket with screws 4, as shown in Figure 2, and the screws should protrude 9-11 mm from the surface of the connection.

ATTENTION: During final assembly of the prosthesis, it is recommended to apply Loctite 241 threadlocker to the threads of screws 4 and respect the following sequence for tightening screws 3 and 4:

Firstly, tighten the screws 4 that are furthest away from slot B with a torque 14 Nm, then screw 3 to 8 Nm and finally tighten the rings 4 that are closest to slot B, as shown in Figure 1, with a torque of 14 Nm.

### RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

### WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.  
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaua - Apdo. de correos 49  
C.P.: 46185 - La Pobla de Vallbona Valencia - España (Spain)  
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33  
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

### WARRANTY

2 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-7H13

EDITION: 05/2015

ISO 13485

ISO 9001