8H02AC

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8H02AC

› 4-prong socket adapter

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The 4-prong socket adapter is used in tibial prostheses. It is fitted to the socket and serves to connect it to other modules of the prosthesis.

TECHNICAL CHARACTERISTICS


Code	8H02AC	
Material	anchor	Aluminium
	adjustment pyramid	Stainless steel
Item weight	73 g	
For patients weighing	up to 100 kg	
Radius of the spherical section, R	47 mm	
Temperature range for use	From -40 °C to +40 °C	
Enables angular adjustment (of the whole prosthesis)	on the frontal and sagittal planes in a range of not less than $\pm 7.5^\circ$	
Warranty period	3 years	

DESCRIPTION OF THE STRUCTURE

The 4-prong socket adapter has been manufactured by machining, ensuring high strength and quality.

The 4-prong socket adapter, position 1, (see Fig. 1) features spherical surface bearing B and an adjustment pyramid, position 2, for connection of the socket to other modules of the prosthesis. This connection enables angular adjustment on the frontal and sagittal planes in a range of not less than $\pm 7.5^\circ$.

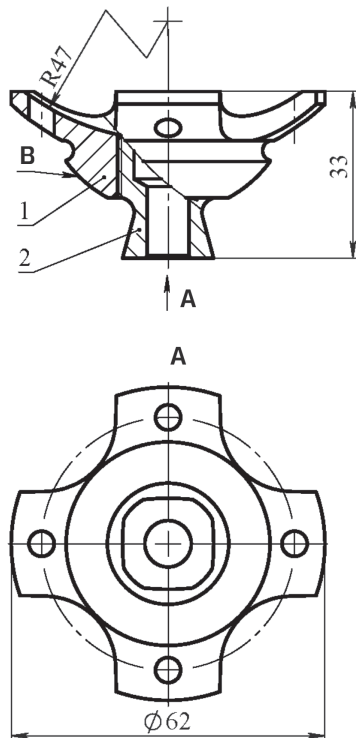The 4-prong socket adapter is compatible with Orliman's modular lower-limb prosthetic systems.

- 1 Anchor
- 2 Adjustment pyramid

FITTING THE ITEM TO THE PROSTHESIS

The 4-prong socket adapter is fitted to the socket by means of lamination.

RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed. Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses. ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49
C.P.: 46185 - La Pobla de Valldona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

WARRANTY

3 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

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APPLICATION

The 4-prong socket adapter is used in tibial prostheses. It is fitted to the liner and serves to connect it to other modules of the prosthesis.

TECHNICAL CHARACTERISTICS

Code	8H02C	8H02T
Housing material	Stainless steel	Titanium
Item weight	92 g	55 g
For patients weighing:	up to 125 kg	up to 150 kg
Radius of the spherical section, R	47.2 mm	
Temperature range for use	From -40 °C to +40 °C	
Enables angular adjustment (of the whole prosthesis)	on the frontal and sagittal planes in a range of not less than $\pm 7.5^\circ$	
Warranty period	3 years	

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The 4-prong socket adapter (see Fig. 1) features spherical surface bearing A and pyramid B for connection of the socket to other modules of the prosthesis. This connection enables angular adjustment on the frontal and sagittal planes in a range of not less than $\pm 7.5^\circ$.

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