# 8H15AK

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# 8H15AK

## › 4-hole socket adapter

### INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,  
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

### APPLICATION

The 4-hole socket adapter is used in femoral or tibial prostheses. The 4-hole socket adapter is laminated in the liner and serves to connect it to other modules of the prosthesis.

### TECHNICAL CHARACTERISTICS

Code	8H15AK
Disc material	Aluminium
Item weight	0.054 kg
For patients weighing	up to 100 kg
Distance between two non-adjacent screws, L mm	36
Temperature range for use	From -40°C to +40°C
Warranty period	3 years


### DESCRIPTION OF THE STRUCTURE

The 4-hole socket adapter features on its edge a slot for fastening during lamination to the socket of the prosthesis.

For connection to other modules of the prosthesis, it features four M6 threaded holes.

The 4-hole socket adapter has been manufactured by machining.

The 4-hole socket adapter is compatible with Orliman's modular lower-limb prosthetic systems.

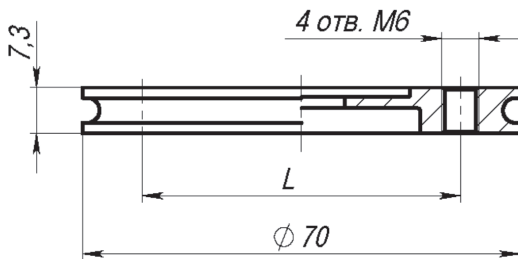### FITTING THE ITEM TO THE PROSTHESIS

The 4-hole socket adapter is fitted to the socket by means of lamination.

During the fitting of the 4-hole socket adapter, it is necessary to control the rotation of the disc for correction of the construction schema of the prosthesis.

During the process of connecting the 4-hole socket adapter to other modules of the prosthesis, the M6 screws must be completely screwed into the base.

During final assembly of the prosthesis, these screws can be sealed with Loctite 241 thread-locker. The tightening torque of the screws is determined by the attachable module (see the Instructions for Use).

### RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed.

Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

### WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses.

ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.  
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliaana - Apdo. de correos 49  
C.P.: 46185 - La Pobla de Vallbona Valencia - España (Spain)  
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33  
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

### WARRANTY

3 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-8H15AK

EDITION: 05/2015

ISO 13485

ISO 9001