8H19 TK

Placa de unión

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8H19

› Connection plate

INSTRUCTIONS FOR USE AND MAINTENANCE

Dear Customer,
Please read the instructions carefully. Make sure that the user understands the instructions for use, paying particular attention to the safety and maintenance warnings.

APPLICATION

The 4-hole socket adapter is indicated for hip or tibial prosthesis receiver sockets. It connects the receiver socket to the 7H08A hole lock.


TECHNICAL CHARACTERISTICS

Code	8H19
Material	Aluminium alloy
Item weight, kg	0.09
For patients weighing	Up to 125 kg
Temperature range for use	from -40°C to +40°C
Warranty period	3 years

DESCRIPTION OF THE STRUCTURE

The 4-hole socket adapter consists of support 1 (Fig. 1), four screws 2 of 30 mm to connect to the 7H08A hole lock and a processing assembly for lamination.

The 4-hole socket adapter has been manufactured by machining, which provides it with high strength and quality.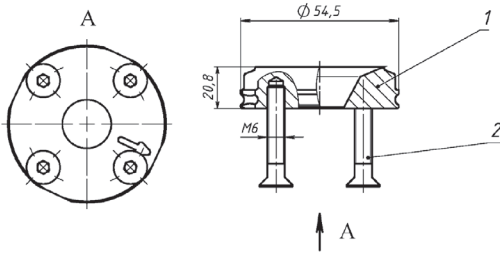


1 Support 1 unit 2 M6 screw, L=30mm 4 units

FITTING THE 4-HOLE SOCKET ADAPTER TO THE PROSTHESIS

Alignment with cover 4 (Fig. 2) is reinforced with bolts 5 in plaster mould 6. (The heads of bolts 5 must not protrude from cover 4). Support plate 2 is connected with technological screws 3 to cover 4 and fixed with screw 1.


The arrow on the plate must be orientated in the anteroposterior direction.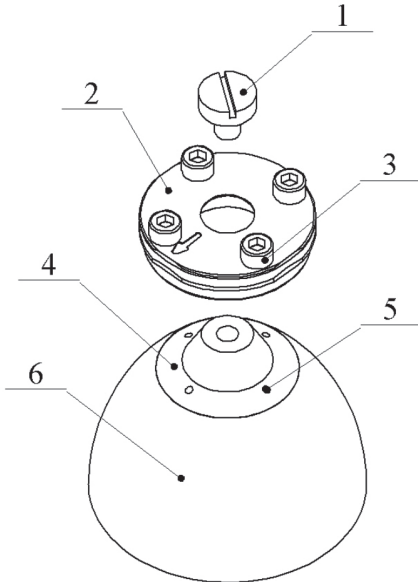

1 Screw 1 unit 4 Cover 1 unit
2 Support plate 1 unit 5 Bolts 4 units
3 Technological screws 4 units 6 Plaster mould

Support plate 2 becomes a receiving capsule after lamination. After lamination, remove the excess resin from the surface of the screws, forming a flat support surface.

Unscrew technological screws 3, screw 1 and cover 4.
Prosthetic socket 1 (Fig. 3) is connected by 7H08A hole lock 3 by means of the 4 screws 4. Check the torque in the Instructions for Use of the 7H08A device.

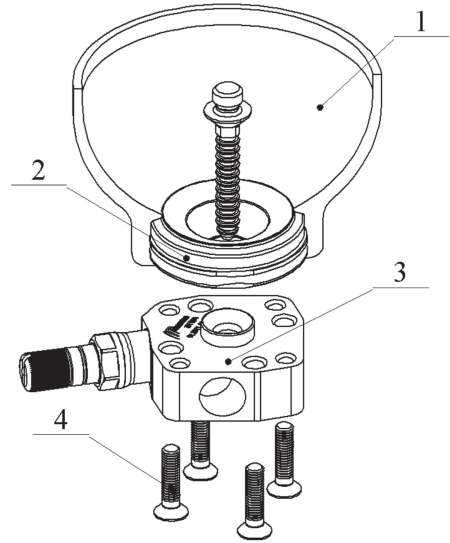1 Prosthetic socket 3 7H08A hole lock
2 Connection plate 4 Screws 4 units

MAINTENANCE

During use or cleaning, do not use talcum powder. Do not put the product into contact with corrosive or abrasive elements such as fresh or salt water, acids, dust, sand or other substances.

At least once a year, have the product checked by an orthopaedic technician for proper operation and wear. If you detect any unusual changes in the product's functioning or hear it making noises, stop using it immediately and notify an orthopaedic technician.

RECOMMENDATIONS - WARNINGS

These products should not be used for purposes other than those described in these instructions. They should only be used by one person and under the conditions prescribed.

Before use, the product must be adapted by an orthopaedic technician.

When disposing of the product and packaging, strictly comply with the refuse disposal regulations of your community.

In knee joints, there is risk of entrapment, use caution.

As the product features metal components, use caution in the case of magnetic resonance.

WARRANTY

These products comply with European Medical Devices Directive 93/42/EEC (Spanish Royal Decree 1591/2009). They have undergone tests in accordance with international standard EN ISO 22523 for prostheses and orthoses and EN ISO 10328 for lower-limb prostheses. ORLIMAN, S.L.U. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose characteristics have been altered through any kind of misuse, breakage or defect. If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Elia - Apdo. de correos 49
C.P.: 46185 - La Pobla de Valbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00 - Exports Tel.: +34 96 274 23 33
E-mail: orto@orliman.com · Exports email: export@orliman.com · www.orliman.com

WARRANTY

3 years from purchase

Reference	Batch no.	Date of purchase
<input type="text"/>	<input type="text"/>	<input type="text"/>

Signature and stamp of the orthopaedic establishment

For the warranty to be valid, it is necessary to complete these details.

LB-I-8H19

EDITION: 05/2015

ISO 13485

ISO 9001