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English**INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,
Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value.
Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATION

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthesis.

FITTING INSTRUCTIONS

The product should be adjusted by your orthopaedic technician or in accordance with doctor's orders.

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

Fitting:

Select the wedges necessary to obtain the required plantar flexion. If it is necessary to remove any of the wedges to adjust the height, always remove the lower section first. The height range is from 0 to 6 cm in intervals of 2 cm. After selecting the desired height, place it into the fabric sleeve and adjust to fit.

PRECAUTIONS

The construction material is flammable.
Do not expose these products in environments where they could ignite. Should this occur, remove them quickly and use appropriate measures to extinguish the flames. The material used is hypoallergenic, however, we cannot guarantee that in some determined cases an allergy may occur. If this happens, remove the product and consult the prescribing doctor. For inconvenience caused by sweating, we recommend using an interface to separate the skin from contact with the tissue. If a rash, irritation or swelling develop, remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and heat build-up.

RECOMMENDATION- WARNINGS

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.
The use of the orthesis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this present sheet of instructions.

MANUFACTURE - CHARACTERISTICS

The material features used in production have been tested and approved and meet all the European quality requirements.
All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthesis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthesis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight etc. During use or during washing do not use alcohols, ointments or solvents. If the orthesis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

**GUARANTEES**

ORLIMAN S. L. guarantees all of its products, provided that their original configuration has not been manipulated or altered. It does not guarantee products that have been altered because of misuse, breakage or deficiencies of any type.

If you find a deficiency or anomaly of any type, please inform the establishment where you purchased it in order to exchange it.

At Orliman, we appreciate the confidence you have shown in our products and we wish you a speedy recovery.

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