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**English****INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with therapeutic properties. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been performed (UNE EN ISO 14971) minimising all the existing risks. The trials have been performed in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**SUITABLE FOR**

**Ref.: CC2600**

- Post-traumatic immobilisation (whiplash).
- Treatment of cervical disc herniation.
- Post-operative protection.

**Ref.: CC2800**

- Cervical spine injuries that require immobilisation of segments C1 to T1 during treatment.
- Post-operative protection.
- Bone metastasis.

**FITTING INSTRUCTIONS**

**THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.**

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

**To fit observe the following aspects:**

**Ref.: CC2600**

Once the size has been selected, fit the orthosis to the patient with the back piece overlapping the front piece, and then fasten with the Velcro; the height of both the occipital and mandibular supports can be adjusted with the buttons. As a precaution, make sure that the orthosis does not compress the glottis and keeps the cervical spine in a neutral position (preventing cervical hyper-extension and flexion), thereby achieving a significant decrease in mobility in the cervical region and reducing pressure on the intervertebral discs. Once the orthosis has been fitted, make sure that the user can swallow normally and that the mandibular support enables the patient to open his/her mouth (2-3 cm) without excessive difficulty.

**Ref.: CC2800**

Once the size has been selected, fit the upper element to the patient with the back piece overlapping the front piece, and fasten with the Velcro. The height of the occipital and mandibular supports can be adjusted with the side buttons. As a precaution, make sure that the orthosis does not compress the glottis and keeps the cervical spine in a neutral position (preventing cervical hyper-extension and flexion). Then fit the lower element and adjust the height to adapt it to the physical characteristics of the patient; to do so, remove the velour covering and slide the front and back plates along the fastening rails. After adjusting the height of the orthosis, fasten the shoulder and side straps and firmly tighten to ensure proper immobilisation of the cervical spine. Finally, make sure that the user can swallow normally and that the mandibular support enables the patient to open his/her mouth (2-3 cm) without excessive difficulty.

The orthosis is designed to be used equally by male and female patients.

**PRECAUTIONS**

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

**RECOMMENDATION-WARNINGS**

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

**MANUFACTURE-CHARACTERISTICS**

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

**RECOMMENDATIONS FOR PRESERVATION AND CLEANING**

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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