SOFY-PLANT gel

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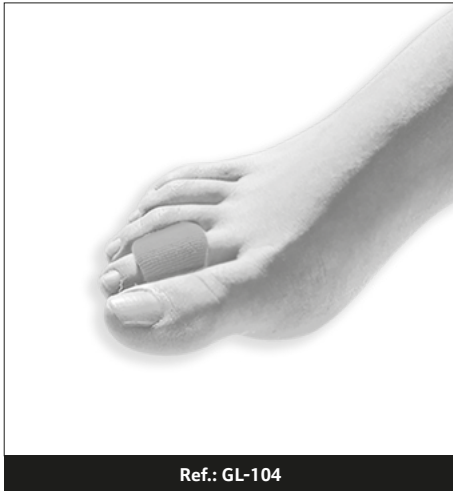
SOFY-PLANT gel

INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATIONS

MD This article is defined as a class I medical device. A Risk Analysis (UNE EN ISO 14971) has been carried out, minimising the existing risks. Tests have been in accordance with European Regulation UNE-EN ISO 22523 on Prostheses and Orthoses.

RECOMMENDED CARE

We recommend that you hand wash product with mild soap and water. Rinse thoroughly. Then air dry product and dust gel with talcum powder without perfume after drying.

Composition: Polymer Gel.

Lined: Polyester.

RECOMMENDATIONS-WARNINGS

Do not place production on an open wound. If irritation, discomfort, or poor circulation occurs, discontinue use and notify your physician immediately. Remove product at least 3 to 4 hours daily to allow skin to breathe. We recommend the correct adjustment, not too tight or too loose as this could produce vascular interruptions. Keep far away from heating source.

Note: if you wear dress shoes it is recommended to spray inside the shoe with a waterproof product.

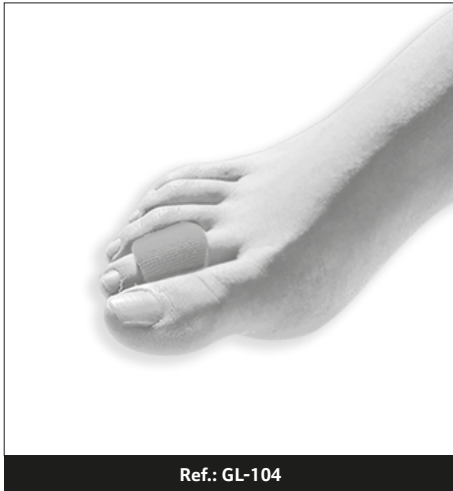
KEEP OUT OF REACH OF CHILDREN.

GUARANTEES

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

EN The manufacturing date is included in the batch number which can be found on the packaging label as **MD**, in the following way: the second and third digits represent the year of manufacture and the fourth and fifth digits represent the month.

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