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**English****INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/EEC (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**RECOMMENDATION- WARNINGS**

Do not place production an open wound. If irritation, discomfort, or poor circulation occurs, discontinue use and notify your physician immediately. Remove product at least 3 to 4 hours daily to allow skin to breathe. We recommend the correct adjustment, not too tight or too loose as this could produce vascular interruptions. Keep fare away from heating source.

Note: if you wear dress shoes it is recommended to spray inside the shoe with a waterproof product.

KEEP OUT OF REACH OF CHILDREN.

**RECOMMENDED CARE**

We recommend that you hand wash product with mild soap and water. Rinse thoroughly. Then air dry product and dust gel with talcum powder without perfume after drying.

Composition: Polymer Gel.

Lined: Polyester.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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