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**English****INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**SUITABLE FOR**

They protect and relieve pain in the metatarsal region, prevent the formation of hardness, absorb shock and reduce calluses.

**FITTING INSTRUCTIONS**

Slip the pad onto the foot, ensuring that the thong is positioned between the big and second toes, and then put on socks, stockings or tights over it. Make sure you have the right size because too much compression can restrict circulation. The pad is easy to fit and can be worn with all types of footwear.

**RECOMMENDED CARE:**

It is recommended to hand wash the product (without scrubbing). Allow to dry in the air. Do not iron. The pads are long lasting.

**RECOMMENDATION- WARNINGS**

Do not place the product over an open wound. In the event of irritation, discomfort or poor circulation, discontinue use. Remove the product for at least 3 or 4 hours a day to allow the skin to breathe. Keep away from heat sources. Keep out of reach of children.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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