Español

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English

INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,
Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value.
Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATION
These products satisfy the European Medical Device Directive 2007/47/CE (Royal Decree 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimizing all the existing risks. The trials have been carried out in accordance with the European regulation UNE 22523 of Prosthesis and Orthesis.

FITTING INSTRUCTIONS
This consists of 2 padded individual supports with a regulation strip to adjust height and rotation.

SUITABLE FOR
Intolerance to breastbone support due to respiratory insufficiency, pigeon chest, breastbone fractures, pectoral asymmetry or others.

POSITIONING
Remove the Jewet frame pectoral support, then secure the pectoral supports in the regulation guides using the screws.
The guides situated in the base of the plate allow for rotation.
Finally, adjust the height of the pads using the regulation guide of the strip

PRECAUTIONS

The constructive material is inflammable.
Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

RECOMMENDATION- WARNINGS

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.
The use of the orthesis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.
For the validity of the guarantee, the issuing establishment should comply with this present sheet of instructions.

MANUFACTURE - CHARACTERISTICS

The material features used in production have been tested and approved and meet all the European quality requirements.
All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthesis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthesis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight etc.
During use or during washing do not use alcohols, ointments or solvents. If the orthesis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

GUARANTEES

ORLIMAN, S. L. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Orliman does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered.
If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.
ORLIMAN, S.L. Appreciates your choice and wishes you a speedy recovery.

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