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**ENGLISH****INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**SUITABLE FOR****OSL3120:**

- Prevention and/or relief of pain in the cervical and upper dorsal paravertebral muscles.

**FITTING INSTRUCTIONS**

In order to achieve the highest degree of therapeutic effectiveness for different disorders and to prolong the service life of the product, it is essential to select the most suitable size for the patient or user (see the size and centimetre equivalence chart on the packaging).

**Follow the fitting instructions below:****OSL3120:**

- 1-Place the collar on the neck with the opening facing forward.
- 2-Tie both tapes for appropriate bracing.

**WASHING AND MAINTENANCE RECOMMENDATIONS****OSL3120:**

Washing the outer cover:

**REMOVE FILLING BEFORE WASHING AND THEN REPLACE IT ONCE DRY.**

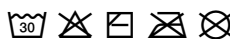
Close the zip before washing.

Machine washable: machine wash at maximum temperature 40°C with neutral detergent. Spin dry at 800-1000 rpm. Can be dried in a dryer, maximum temperature 60°C. Ironing, maximum temperature 150°C. Do not use bleach. Do not dry clean.

Washing the inner cover with the filling:

Machine washable: machine wash with delicate programme and tepid water (30°C) with neutral detergent. Spin dry gently (400 rpm). Leave to dry at room temperature. Do not hang up, iron or expose to direct heat sources like stoves, heaters, radiators, direct sunlight, etc. Do not use bleach. Do not dry clean.

When using or cleaning, do not use bleach, alcohol, ointments or liquid solvents. If the product is not well wrung out, detergent residue can remain and cause irritation to the skin and damage to the product.

Store in a dry place at room temperature.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

**COMPOSITION**

> Outer cover:

- Fabric: 70% bamboo, 30% Lycra.
- Zip closure system: 100% polyester.

> Anatomical collar: viscoelastic polyurethane foam - density 60 kg/m<sup>3</sup>.

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