Español

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English

INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATION

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

SUITABLE FOR

Orliven® Sport

· Better venous return to improve muscle recovery, reduce fatigue during exercise and decrease recovery periods.

Orliven® Travel

· Improved venous return, thus making the sock suitable for all users who make journeys in which they have to remain seated for many hours.

Orliven® Relax Diabetic / Orliven® Daily Diabetic

- Diabetic foot.
- Candidiasis.
- Mycosis.
- Eczema.
- Preventing the proliferation of bacteria and fungi (bacteriostatic).

FITTING INSTRUCTIONS

Orliven® Sport / Orliven® Travel

- Before putting on the sock, make sure your foot is completely dry.
- Turn the sock inside out as far as the heel.
- Insert your foot as far as the heel.
- Pull the sock up over the lower part of your leg.
- Make sure the fabric is evenly distributed.
- Do not fold over the top of the sock.

PRECAUTIONS - WARNINGS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

MANUFACTURE - CHARACTERISTICS

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Washing instructions:

- Wash daily.
- The socks are machine washable using the delicates program at 30°C / Or wash by hand.
- Only use detergent for delicate clothes. Do not use fabric softener because it can close the pores and matt the fibres together.
- Never dry the socks in the sun or on a radiator. Dry horizontally in the shade or hang them out.
- Do not use chemical cleaning agents or bleach and do not iron.

GUARANTEES

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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