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English

INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATION

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

SUITABLE FOR

Girdle-trousers of this type are indicated for persons with flaccid muscle tissue with prominent pendulous abdomen, sagging glutei and soft thigh musculature associated with lower back pain incisional hernias, post-abdominal wall surgery, arthritis and rheumatic problems, etc. in which cosmetic appearance is important as well as compression which gathers and contains all the associated musculature, obtaining elevation of the lower abdomen and creating an upwards compression, preventing chafing of the inner thighs.

FITTING INSTRUCTIONS

THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

To fit observe the following aspects:

It is recommended to be used with underwear so as to obtain greater comfort and hygiene.

With the opening systems open, place each leg into its corresponding trouser leg.

Then lift the girdle-trousers up to the pubic area, gathering the glutei. Raise the rest of the girdle as high as possible, up to about just below the ribs, and close the girdle starting below and going upwards, starting with the hook-and-eye system, then closing the zip.

Adaptation of the lumbosacral support:

The lumbosacral support is an elastic element in the back zone of girdle-trousers. At both ends it has 4 rows of hooks which allow us to regulate the pressure of the elastic lumbosacral system. On the front (abdominal), there are 2 tabs which on their ends, a row of male hooks. Pull the back elastic bands toward the abdomen, fastening them with the hooks on the desired row in order to obtain the necessary compression, always starting from the lowest, and going to the highest hook. Carry out this procedure on both sides.

Attachment of the lumbar cushion:

The panty girdle has an anatomically designed lumbar pad.

Before putting in place, it needs to be adapted to the patient's lumbar area. Once this has been done, remove the astrakhan Velcro straps and position the fastener area on the velour provided for this purpose.

The widest area should be positioned upwards and the narrowest edge downwards. For proper positioning, the Velcro face should be attached to the rear internal side of the girdle, positioned in accordance with the female Velcro and properly aligned.

The plate should be adapted to the patient's lumbar curve, which we can do using our hand. Take care to ensure that the Velcro fastener does not come into contact with the external part of the orthosis, in order to avoid chafing with the fabric.

PRECAUTIONS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat. The product contains natural rubber latex and may elicit allergic responses in individuals who are sensitised to latex.

RECOMMENDATION-WARNINGS

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

MANUFACTURE-CHARACTERISTICS

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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