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**English****INSTRUCTIONS FOR USE AND PRESERVATION**

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

**FEATURES**

Ultra slim flexible silicone. The insoles offer support for your foot and relieve the sensitive areas on the balls of your feet and the heel area. Polyester-lined antibacterial, hypoallergenic surface.

**SUITABLE FOR**

Indicated for pained or tired feet. Calcaneal spurs and/or metatarsalgia. Relief of ligaments and tendons (tendonitis). Plantar fasciitis. Pain and rheumatism (back, hip, ankle, knee). Sensitive feet, and for playing sports.

Composition: 100% ultra slim silicone. Polyester lining.

**FITTING INSTRUCTIONS**

- 1-Choose the proper insole for your shoe size.
- 2-Insoles whose size does not fit the shoe will be cut from the frontal part.
- 3-Insert the marked insoles (R=right, L=left), into the corresponding shoe.
- 4-Warning: It may require a period of 2-3 days to get used to the insole until it feels comfortable.

**PRECAUTIONS**

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

**RECOMMENDATION- WARNINGS**

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

**MANUFACTURE - CHARACTERISTICS**

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

**RECOMMENDATIONS FOR PRESERVATION AND CLEANING**

The insoles may be washed by hand with water and pH-balanced soap at 30°-40°.

**GUARANTEES**

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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