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## English

### INSTRUCTIONS FOR USE AND MAINTENANCE

Dear customer,  
Thank you very much for choosing this Orliman product. You have purchased a high-quality health-care product.  
Please read the instructions carefully. If you have any queries, please consult your doctor, orthopaedic specialist or our customer service department. Orliman thanks you for choosing its product and wishes you a speedy recovery.

### REGULATIONS

These products conform to European Directive 93/42/EEC concerning Medical Devices (Royal Decree 1591/2009). A risk analysis has been performed (ISO 14971) to ensure all existing risks have been minimised. Tests have been conducted in accordance with ISO 22523 - External limb prostheses and external orthoses.

### INDICATIONS

As an accessory for osteoporosis bodysuits, providing greater rigidity and immobilisation in the dorsolumbar area of the corset.

### PLACEMENT INSTRUCTIONS

THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.

To place the cushion in the correct position, follow these steps:

Insert the spinal support into the pocket at the back of the bodysuit.

If the support needs to be shortened, remove one of the middle sections. Cut the aluminium splints to fit the new length and insert the support into the pocket of the bodysuit.

### PRECAUTIONS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

### RECOMMENDATION- WARNINGS

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

### MANUFACTURE - CHARACTERISTICS

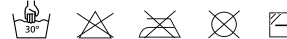
The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

### RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap.

To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

### GUARANTEES

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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