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English**INSTRUCTIONS FOR USE AND MAINTENANCE**

Dear customer,
 Thank you very much for choosing this Orliman product. You have purchased a high-quality healthcare product.
 Please read the instructions carefully. If you have any queries, please consult your doctor, orthopaedic specialist or our customer service department. Orliman thanks you for choosing its product and wishes you a speedy recovery.

REGULATIONS

These products conform to European Directive 93/42/EEC concerning Medical Devices (Royal Decree 1591/2009). A risk analysis has been performed (ISO 14971) to ensure all existing risks have been minimised. Tests have been conducted in accordance with ISO 22523 - External limb prostheses and external orthoses.

INDICATIONS

A hand-wrist extension attachment for the finger immobilising glove splint to extend immobilisation to the wrist joint.

PLACEMENT INSTRUCTIONS

THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.

To place the cushion in the correct position, follow these steps:

Attach the interior surface of the accessory to the dorsal surface of the glove splint by means of the Velcro. Secure with the red fastening strap. Finally adjust the strap end completely wrapping the wrist joint.

PRECAUTIONS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

RECOMMENDATIONS-CAUTION

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

MANUFACTURE-FEATURES

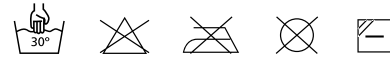
All features of the materials used in manufacture have been tested and certified, and comply with European quality standards.

All products listed are made from the highest quality materials and offer excellent comfort and quality of use.

WASHING AND MAINTENANCE RECOMMENDATIONS

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap.

To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

**GUARANTEES**

ORLIMAN, S.L. guarantees all of its products, provided that they have not been tampered with or altered from their original form. The company does not guarantee products whose features have been altered through any kind of misuse, breakage or defect.

If you notice any defect or abnormality, notify the supplier immediately in order for the product to be replaced.

ORLIMAN, S.L. thanks you for choosing its product and wishes you a speedy recovery.

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